

PERTHESE™ Esthea Breast Implants

Cohesive silicone gel-filled breast implants



PERTHESE™ Esthea Breast Implants

Micro-textured cohesive silicone gel-filled breast implants

Using PERTHESE™ Breast Implant technology, PERTHESE™ Esthea range is composed of 7 models in various sizes. PERTHESE™ Esthea Breast Implants are the result of research and development targeting the most natural aesthetic results.

> Safety

- The PERTHESE™ Esthea Breast Implant is a sterile gel-filled implant made of a trilaminar silicone envelope.
- The silicone envelope consists of an internal and external layer made with a high mechanical resistance medical grade silicone elastomer and an intermediate silicone barrier layer to significantly reduce gel bleed. The envelope is filled with a soft and cohesive silicone gel, very close to the normal breast's consistency. The elongation at break of the shell is greater than 450%.
- Each implant has engraved on the patch the manufacturer's identification, the serial number and the volume (in cc).
- The base of the PERTHESE™ Esthea Breast Implant has orientation marks that make it possible to facilitate the positioning of the prosthesis in the pocket.
- The PERTHESE™ Esthea Breast Implant is provided sterile within a rigid double blister package in order to ensure more security and convenience. The prosthesis is sterilised with ethylene oxide using a validated sterilisation cycle.

> Tolerance


The micro-textured shell of PERTHESE™ Esthea Breast Implants provide a soft envelope while allowing a non-aggressive disorganisation of the collagen structure.

The micro-texture carries several guarantees:

- Reproducible texture, identical from one implant to the other
- Non-alterable physical features of the envelope.

> Warranty

All PERTHESE™ Silicone Gel-filled Breast Implants are covered by PERTHESE™ Quality Assurance Program warranty or PatientCare Promise limited warranty†

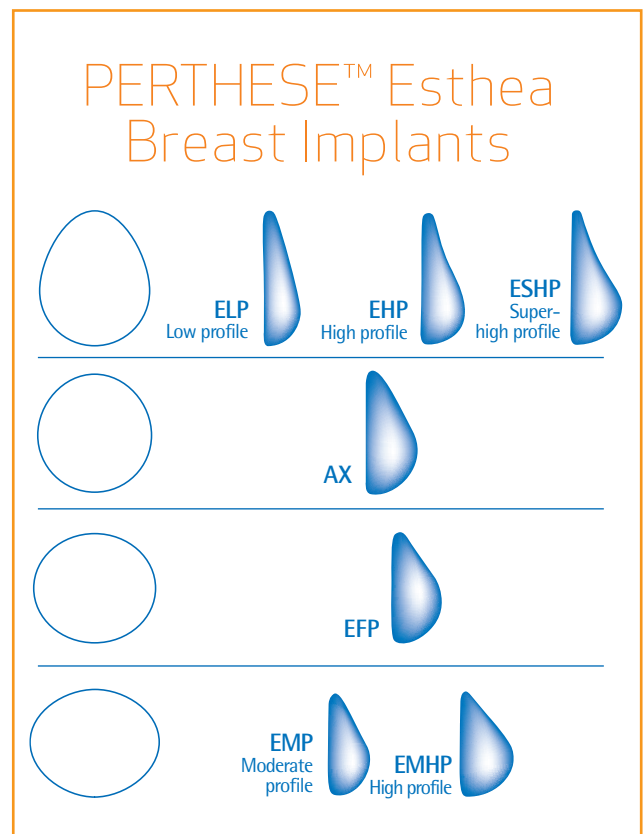

† In case of eligibility, depending on the date of surgery, the implant will fall under one of the above warranties.





PERTHESE™ Esthea Breast Implants ELP/EHP/ESHP

These implants have a similar base but different projections.


- > The shape of the ELP, EHP and ESHP Breast Implants respects the curves of the breast and gives it a very natural appearance.
- > For each base, 3 projections are proposed, which make it possible to respond to the individual wishes of the patients and correct potential asymmetries.

A "Tracer" is available to adapt the shape of the pocket to the dimensions of the implant.

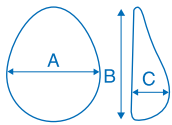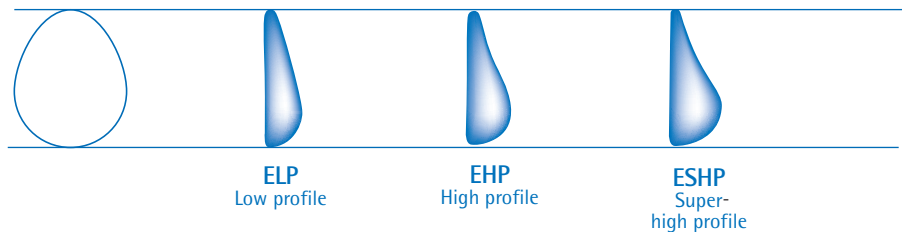


Width cm "A"	Height cm "B"	Projection cm "C"	Volume (cc)	Reference
10,0	12,5	3,5	190	ELP 190
		4,0	225	EHP 225
		4,6	235	ESHP 235
11,1	13,8	3,6	240	ELP 240
		4,2	275	EHP 275
		4,8	295	ESHP 295
11,7	14,0	3,9	270	ELP 270
		4,7	325	EHP 325
		5,3	355	ESHP 355
12,3	14,4	4,2	310	ELP 310
		5,1	375	EHP 375
		5,6	415	ESHP 415
12,6	14,8	4,5	360	ELP 360
		5,4	425	EHP 425
		5,9	475	ESHP 475

PERTHESE™ Esthea Breast Implants AX

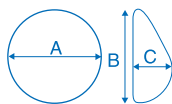


Width cm "A"	Height cm "B"	Projection cm "C"	Volume (cc)	Reference
10,6	10,8	4,3	190	AX 190
10,9	11,2	4,4	210	AX 210
11,5	12,2	4,6	240	AX 240
11,9	12,5	4,8	260	AX 260
12,1	12,8	4,9	285	AX 285
12,6	13,2	4,9	300	AX 300
12,9	13,7	4,9	340	AX 340
13,3	14,1	5,0	380	AX 380
13,6	14,2	5,1	400	AX 400
13,9	14,7	5,2	450	AX 450
14,5	15,2	5,3	480	AX 480
14,9	15,6	5,3	500	AX 500

PERTHESE™ Esthea Breast Implants EFP

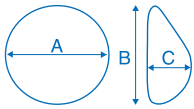

Width cm "A"	Height cm "B"	Projection cm "C"	Volume (cc)	Reference
8,1	7,5	3,4	95	EFP 095
8,6	8,1	3,6	115	EFP 115
9,2	8,5	3,8	140	EFP 140
10,2	9,2	4,1	180	EFP 180
10,7	9,7	4,3	210	EFP 210
11,2	10,2	4,5	250	EFP 250
11,7	10,7	4,7	285	EFP 285
12,2	11,3	4,9	325	EFP 325
12,7	11,8	5,1	370	EFP 370
13,1	12,2	5,3	415	EFP 415
13,6	12,6	5,5	470	EFP 470
14,0	13,1	5,7	520	EFP 520
14,6	13,5	5,9	575	EFP 575
15,1	14,0	6,1	640	EFP 640
15,5	14,2	6,3	710	EFP 710

PERTHESE™ Esthea Breast Implants EMP/ EMHP

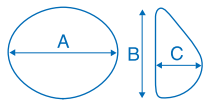

Width cm "A"	Height cm "B"	Projection cm "C"	Volume (cc)	Reference
8,1	6,8	3,2	80	EMP 080
		3,5	90	EMHP 090
8,6	7,3	3,5	100	EMP 100
		3,8	110	EMHP 110
9,2	7,7	3,6	120	EMP 120
		4,0	130	EMHP 130
10,2	8,5	3,9	145	EMP 145
		4,3	170	EMHP 170
10,7	9,1	4,0	165	EMP 165
		4,5	195	EMHP 195
11,2	9,6	4,3	200	EMP 200
		4,7	235	EMHP 235
11,7	10,1	4,5	230	EMP 230
		4,9	270	EMHP 270
12,2	10,6	4,6	265	EMP 265
		5,1	320	EMHP 320
12,7	11,1	4,9	305	EMP 305
		5,3	360	EMHP 360
13,1	11,6	5,0	345	EMP 345
		5,5	405	EMHP 405
13,6	12,0	5,2	390	EMP 390
		5,7	455	EMHP 455
14,0	12,5	5,5	435	EMP 435
		5,8	490	EMHP 490
14,6	13,0	5,7	475	EMP 475
		6,0	545	EMHP 545
15,1	13,3	5,9	535	EMP 535
		6,2	625	EMHP 625
15,5	13,6	6,1	585	EMP 585
		6,4	680	EMHP 680

As slight variation in size and weight can occur with these products, not all prostheses will have the exact measurements and weight indicated above.



> Indications

- Contour reconstruction and size augmentation following a subcutaneous mastectomy for benign disease, e.g., chronic cystic mastitis, or for reconstruction and augmentation for correction of defects following radical mastectomy procedures.
- Unilateral or bilateral mammary augmentation or reconstruction to surgically correct various congenital or acquired anomalies such as amastia, aplasia, hypomastia, hypoplasia or following a reduction mammoplasty or for cosmetic purposes.

Criteria for patient selection are the sole responsibility of the surgeon.

> Precautions for use

Precautions for use should be considered by the surgeon prior to the use of a PER THESE™ Esthea Breast Implant:

- No breast cancer in progress at the time of implantation, even if a breast reconstruction can be considered in the case of a controlled metastatic syndrome, the prerequisite being that the benefits to the patient must be weighed up against the consequences of the surgical procedure and its non-interference with the required carcinologic treatments.
- If the patient exhibits a lack of skin, it is the sole responsibility of the surgeon to decide on the use of an expansion technique or a tissue graft.
- Unwillingness on the part of the patient to accept the possibility of multiple surgeries for revision, which could be related to a lack of understanding, inappropriate motivation, or a history of repeated failures following cosmetic augmentations or subcutaneous mastectomy.
- From a general point of view, the health condition of the patient should be considered in view of the type of surgical procedure and the required anaesthesia.
- The patient should be informed of the existence of alternatives to breast implant surgery and/or to the availability of non-silicone gel-filled breast implants.

Each patient must be evaluated by the surgeon to balance the risk/benefit to the patient.

> Contraindications

PER THESE™ Esthea Breast Implants should not be implanted in a patient who has:

- a general or localised infection of the mammary sphere,
- an ongoing pregnancy,
- a history of autoimmune disease.

> Warranty

PEROUSE PLASTIE warrants that reasonable care in selection of materials and methods of manufacture was exercised in the fabrication of this product.

PEROUSE PLASTIE shall not be liable for any incidental or consequential loss, damage, or expense, directly or indirectly arising from the use of this product.

The foregoing warranties, as conditioned and limited, are in lieu of and exclude all other warranties not expressly set forth herein, whether express or implied by operation of law or otherwise.

PEROUSE PLASTIE neither assumes nor authorises any other person to assume for it any other or additional liability or responsibility in connection with this product.

PEROUSE PLASTIE intends that these products should be used only by physicians having appropriate training in plastic, reconstructive and aesthetic surgery techniques.

PER THESE® Esthea Breast Implants is a trademark of PEROUSE PLASTIE.

> Contact Information

Corporate Headquarters

201 Mentor Drive
Santa Barbara, CA 93111 USA
Tel: 1-805-879-6000
www.mentorwllc.com

Operations Europe, Middle East, Africa

Germany, Austria & Switzerland

Mentor Deutschland GmbH
Tel: +49 811 600 50 0
Fax: +49 811 600 50 11

Italy, Greece & Israel

Johnson & Johnson Medical S.p.a.
Tel: +39 06 911 94 500
Fax: +39 02 644 10 563

France & Benelux

ETHICON France, S.A.S.
Tel: +33 1 55 00 22 45
Fax: +33 1 55 00 28 11

Croatia, Macedonia, Serbia, Slovenia & Bulgaria

Johnson & Johnson S.E. d.o.o.
Tel: +386 1 40 11 800
Fax: +386 1 40 11 801

Poland, Lithuania, Latvia & Estonia

Johnson & Johnson Poland sp. z o.o.
Tel: +48 22 237 83 05
Fax: +48 22 237 83 53

South Africa

Johnson & Johnson Medical (Pty) Ltd
Tel: +27 11 265 1135
Fax: +27 11 265 1114

Middle East, Egypt & Pakistan

Johnson & Johnson Middle East Inc
Tel: +971 4 429 72 00
Fax: +971 4 299 43 30

Spain & Portugal

Mentor Medical Systems Iberica, S.L
Tel: +34 91 562 2700
Fax: +34 91 728 3367

United Kingdom & Ireland

Johnson & Johnson Medical Ltd
Tel: +44 (0)800 973 082
Fax: +44 (0)134 486 4122

Russia & CIS

Johnson & Johnson LLC
Tel: +7 495 580 7777
Fax: +7 495 580 7878

Czech Republic, Slovak Republic, Hungary & Romania

Johnson & Johnson s.r.o.
Tel: +420 227 012 371
Fax: +420 227 012 300

Sweden, Denmark, Norway, Finland & Iceland

Johnson & Johnson AB
Tel: +46 8 626 2200
Fax: +46 8 754 5850

Turkey

Johnson & Johnson Medical Turkey
Tel: +90 216 538 21 21
Fax: +90 216 538 21 30

Maghreb & Sub Saharan Africa

Johnson & Johnson Professional Export
Tel: + 44 (0)1506 594 500
Fax: + 44 (0)1506 594 731

